



## - Technical Sheet HIPPO MB Modular Postural Pad for Hippotherapy -

<u>Manufacturer</u> identification	Pro Medicare S.r.l.         Via Montagna Z.I. Lotto 41         72023 Mesagne (Br)         P IVA 01803920741         Tel. 0039 0831 777840 Fax 0039 0831 730739         Website : www.promedicare.eu
Device Commercial Name	НІРРО МВ
UDI-DI BASIC	805571347HIPPOEM
Device Code	<b>D02-701-1</b> underbelly closure on the right <b>D02-702-1</b> underbelly closure on the left
Ministry/Repository Registration Number	2149978
Code of Classification (CND)	Y180839
Medical Device Class	The HIPPO MB modular postural pad is CE marked according to EU Regulation 2017/745 and it is a non sterile Class I Medical Device.
Intended use	HIPPO MB is a comfortable modular postural pad, also equipped with a girth and flexible handle suitable for the execution of equestrian rehabilitation treatment. The HIPPO MB modular postural pad was designed to be used during a session of Horse Assisted Therapy, also called Equestrian Rehabilitation, Hippotherapy. The HIPPO MB modular postural pad, is presented with a multiplicity of postural elements that can be customized according to the needs of the end user and the resulting postural adaptations with reference to both supine and prone as well as sitting positions. The entire device is designed to allow: - the homogeneous distribution of pressures; - the containment and postural alignment; The combined and harmonious use of the lower and upper elements together with the postural elements depending on the position, sitting or lying (supine/prone) is designed to promote posturally correct, safe and comfortable support for the end user.

	The HIPPO MB modular postural pad consists of:         -       Lower and upper element;         -       Head positioning system;         -       Pelvis positioning system kit;         -       Push trunk positioning system kit;         -       Confortable Trunk positioning system kit;         -       Abductor insert;         -       Femoral Condyles positioning guides kit;
	<ul> <li>Hand grips kit;</li> <li>Knee bend positioning system kit;</li> <li>Crupper.</li> </ul> DIMENSIONS: PAD Dimensions: Length x Width: 105x90 [cm]/41.33x35.43 [in] WEIGHT: 4,5 kg/9.92 [lb] MAXIMUM LOAD: 60 kg/132.2 [lb] please refer to the relevant order form for details of the standard configuration and related accessories/spare parts. PERFORMANCE AND DURABILITY:
<u>Technical features</u>	The duration of maintenance of performance, under safe conditions, of these devices is defined as 2 years (padding) valid under normal conditions of use and proper maintenance in accordance with the manual. <b>MATERIALS:</b> The entire system is made of polymeric material and combines different characteristics of density, elasticity, lift and shape memory; the whole is covered with different fabrics with combined composition: - Nylon/Lycra in the case of the upper element; Delwrmide Electere Gilicene Nylon (how for the lower element)
	<ul> <li>Polyamide, Elastane, Silicone, Nylon/Lycra for the lower element;</li> <li>PU/PES in the case of the postural element coverings. These coverings have the following characteristics: <ul> <li>antibacterial, non-cytotoxic, non-irritating</li> <li>resistant to virus penetration</li> <li>antifungal, fireproof, biocompatible</li> <li>OEKO-TEX<sup>®</sup> certified, latex-free</li> <li>breathable, waterproof, washable and sanitizable</li> </ul> </li> </ul>
	MANTEINANCE AND CLEANING: The cover of the upper and lower element, being removable, can be hand-washed with warm water (30°C) and then air-dried. The covering of the postural elements can be sterilized by washing it at 95° and sanitized with the following detergents/disinfectants: – PERSIL EXPERT coldzyme/HENKEL – 70% etanol – PERFORM/Schuelke
	<ul> <li>TPH PROTECT/Schuelke</li> <li>MIKROZID AF liquid/Oktalpharma</li> <li>INCIDIN PLUS/Ecolab</li> <li>DESCOGEN liquid/Antiseptica</li> <li>MANORAPID/Antiseptica</li> <li>PLIVASEPT/Pliva</li> <li>SODIUM HYPOCHLORITE-BASED PRODUCTS CANNOT BE USED</li> </ul>
	<ul> <li>DO NOT USE DRYERS TO HAVE A LONGER PRODUCT LIFE.</li> <li>PACKAGING:</li> <li>It comes inside its PAD bag with the postural elements placed in the plastic case and individually packed in cardboard.</li> </ul>

